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Instructions for use

Please read this information carefully because it will help you use HYLO FRESH® properly.

HYLO FRESH®

Sodium hyaluronate 0.3 mg/ml

For relief of irritated eyes

Sterile, phosphate-free eye drops without preservatives

What is HYLO FRESH®?

HYLO FRESH® is a sterile preservative-free solution for ocular use.

Composition:

1 ml HYLO FRESH® contains 0.3 mg sodium hyaluronate, Euphrasia-original mother tincture, boric acid, borax and water for injections.

What is HYLO FRESH® used for?

For relief of irritated eyes.

When should you use HYLO FRESH®?


HYLO FRESH® is used for the relief of irritated eyes. Irritated and dry eyes can result from environmental stress such as intensive computer-based activity, dry climate, draught, increased solar radiation, air pollution caused by dust or pollen or wearing contact lenses. For hundreds of years, preparations of eyebright (*Euphrasia officinalis*) have traditionally been used for the treatment of conjunctival irritation with red and burning eyes, increased tear secretion, swelling and red eyelids or eyelid margins. Moreover, HYLO FRESH® contains sodium hyaluronate which increases the viscosity of the moisturizing solution making it thicker than water. It therefore remains longer on the ocular surface. The application of the eye drops soothes irritated eyes and eyelids and provides them with a tear-like moistening effect.

HYLO FRESH® treats all symptoms caused by drying of the ocular surface, e.g. due to insufficient physiologic tear production by the body. HYLO FRESH® also moistens the eyes of contact lens wearers.

Being preservative-free, HYLO FRESH® is generally very well tolerated. Additionally, HYLO FRESH® is phosphate-free, so that possible complications due to the formation of corneal deposits are avoided.

How should HYLO FRESH® be applied?

Please do not use HYLO FRESH® if any part of the seal on the folding carton is broken or damaged before using for the first time (☹).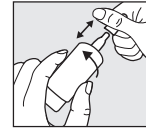


Step 1:

Remove cap before use.

To do this, hold the cap above the bottle. With your other hand, pull away the bottle from the cap with a rotary movement.

Before the first application please turn the HYLO FRESH® bottle with its tip down and press the bottom of the bottle repeatedly until the first drop appears at the nozzle. Now the bottle is ready for further use.

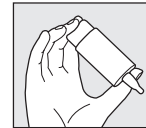
Step 2:

Hold the bottle upside down and place your thumb on the shoulder of the bottle and the other fingers on the base of the bottle.


Step 3:

Support the hand holding the COMOD®-bottle with your other hand as shown.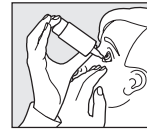

Step 4:

With your head leaning back slightly, gently pull down your lower eyelid. Press down the base of the bottle vigorously and quickly. This activates the mechanism to release one drop. Because of the special COMOD® valve system, size and speed of each drop remain constant regardless of the pressure applied to the base of the bottle. Slowly close your eyes to allow the fluid to spread evenly on the eye surface.

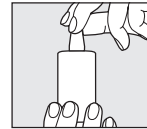Step 5:

Replace cap immediately and carefully after use.

Ensure that the dropper tip is dry.

During use, please avoid any contact between the tip of the bottle and your eye or your skin.

Each bottle releases 10 ml of solution which is equivalent to approximately 300 drops. For technical reasons a small amount of solution will remain in the bottle.

The duration of use of HYLO FRESH® is not limited.

HYLO FRESH® is only for you, even people with similar symptoms should not use your eye drops.

What else do you need to know?

If you use any other eye drops there should be at least 30 minutes between the applications, and HYLO FRESH® should always be used last. Eye ointments should, however, always be administered after using HYLO FRESH®.

Because it is preservative-free, HYLO FRESH® is well tolerated even when used over a longer period of time. In very rare cases hypersensitive reactions like burning or excessive tearing have been reported which stopped immediately when the use of HYLO FRESH® was discontinued.

HYLO FRESH® may be used while wearing contact lenses. After inserting contact lenses, waiting for about 30 minutes before applying the eye drops is recommended. In doing so you avoid intolerance reactions which may occur due to interactions between lens care products and HYLO FRESH®.

Based on the available evidence HYLO FRESH® may be used in children, pregnant and breast-feeding women.

How often and for how long can you use HYLO FRESH®?

HYLO FRESH® should be used according to your condition. Generally, you instil one drop of HYLO FRESH® three times a day into each eye. If necessary, you may use it more frequently. If you use HYLO FRESH® more frequently (e. g. more than 10 times per day) please consult your eye doctor. HYLO FRESH® is suitable for long-term treatment. Nevertheless, as with all complaints, you should consult your eye doctor if symptoms persist.

When should you not use HYLO FRESH®?

Do not use HYLO FRESH® if you are hypersensitive to one of the ingredients.

What you need to know about storing HYLO FRESH®?

Do not store above 25 °C (↘^{25 °C}).

HYLO FRESH® can be used for 6 months after opening.

Do not use beyond the expiry date (📅).

Keep out of the reach of children.

The following packaging sizes are available:

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STERILE A

Medical device

Your opinion about our product is relevant for us!
We are looking forward to get your feedback via mail
or via www.ursapharm.info/hylofresh.

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URSAPHARM Arzneimittel GmbH,